

Cyber Blade™



THE 1st INTEGRATED CORDLESS DISPOSABLE MORCELLATOR

MARKING A NEW ERA IN BPH SURGERY



No maintenance
No service



No time-demanding
and costly reprocessing



No delays due to reusable
morcellator availability



Complete package with everything
needed for a single procedure



No cross contamination risk
among cases



Always a new perfectly sharp
and high-performance blade
morcellator availability

| Overview



CORDLESS AND WITHOUT CONSOLE

- ALL functions are managed by hand control
- light and ergonomic handpiece
- no need for motor and vacuum consoles
- complete freedom to operate

STERILE SINGLE-USE DEVICE


- neither preparation nor sterilization required – no cleaning, no risk of contamination
- ready to use; saves time and money

REUSABLE INTERNAL UNIT

- high tech internal control unit (no need for motor station)
- completely rechargeable, reusable batteries



| How It Works

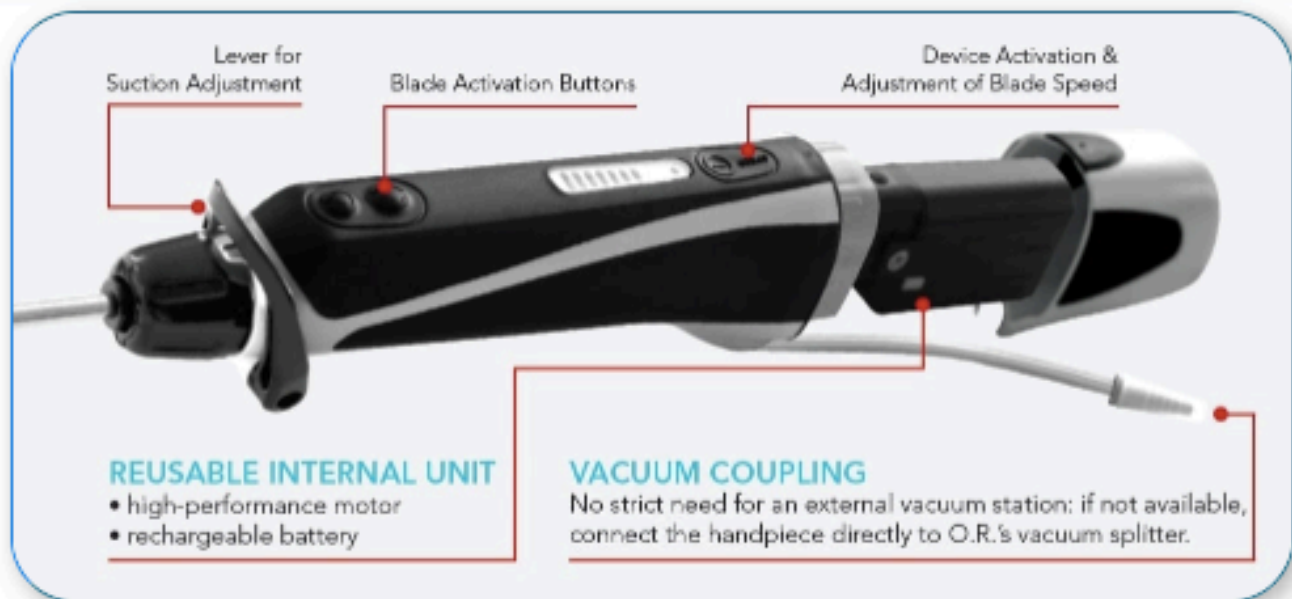

- Single use handpiece + blade
- No additional system needed
- Morcellation speed – (faster Vs Piranha)
- Rotation Speed – up to 6000 rpm
- Wireless system
- No stop-time for 2nd Patient
- Always perfect efficiency for each patient
- Blade activated by simply pushing either button; by holding/releasing the finger interrupts the blade rotation – OR by “on” setting until pressed “off”
- Blade speed has 7 settings – recommended is 2 or 3

| Battery-Motor



7 SPEED STEPS

- visualization by LED graduation scale

UP TO 80 MINUTES IN ACTIVE MODE

- More than 1 treatment without recharge

RECHARGING TIME – 70 MINUTES (full recharge)

2 WORKING MODES

- Working continuously (Emergency-Stop with all buttons)
 - Working only with activation button pressed
-

| Charging Station



4 SLOTS AVAILABLE

- No problem for multiple consecutive procedures

FAST RECHARGING PROCEDURE

- No connectors presence

SIMULTANEOUS CHARGING

WIDE-RANGE POWER SUPPLY

- 110/220VAC
 - Removable Power Cord (no socket adaptor issues)
-